

ATOM MEDICAL



REF: AMI-160824-01
DATE: August 24, 2016

SPECIFICATION CHANGE NOTICE

PRODUCT: ATOM Transcapsule V-808
CHANGE: Description in the operation manual/service manual

DETAILS OF CHANGE:

Please be advised that the description about the "faulty battery alarm" in the operation manual and service manual of V-808 was changed as follows:

<Old description>

12-15. Faulty Battery Alarm

If a charging current keeps on flowing even after a specified time has elapsed while the batteries are being charged, or if a battery charge level falls below a specified level after charging a battery, about half the corresponding battery charge level indicator and the reset indicator will flash.

Action to take : The batter may have deteriorated in quality.

Alarm reset : Press the reset switch. The reset indicator will stop flashing and remain on. Then press the reset switch again. (The batteries will begin to be charged. If the alarm should occur again, the battery may be faulty.)

<New description>

12-15. Faulty Battery Alarm

If a charging current keeps on flowing even after a specified time has elapsed while the batteries are being charged, or if a battery charge level falls below a specified level after charging a battery, about half the corresponding battery charge level indicator and the reset indicator will flash.

Action to take : The batter may have deteriorated in quality. Replace the old battery.

WARNING



Do not press the reset switch if the battery has deteriorated. The battery may generate heat, explode or catch fire.

SCHEDULE OF CHANGE:

Immediately

CHANGE OF DOCUMENTS:

- Operation manual: Yes No
- Service manual: Yes No
- Spare parts list : Yes No
- Distributor Price List: Yes No